

Installation Instructions — AED Plus Aviation Battery Assembly

Press the AED Plus Aviation Battery Assembly against the back of the battery well and then press down (see Figure 1), taking care to align the pins on the underside of the battery assembly with the connector socket in the battery well (see detail, Figure 2).

Figure 1.

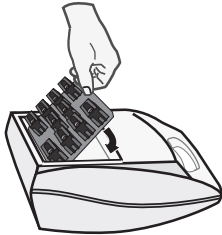
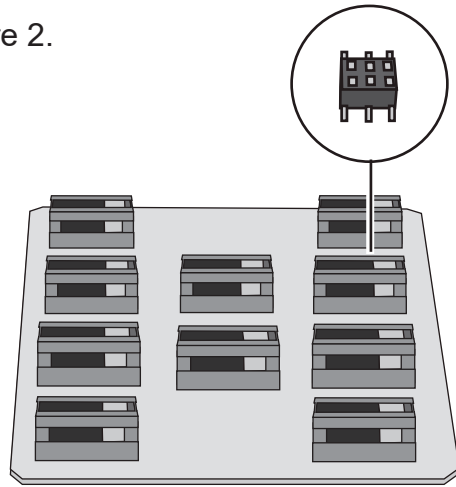


Figure 2.



Caution: Do not press battery assembly against the *front* of the battery well when installing it (see Figure 3) — doing so may bend the connector pins and render the battery assembly unusable.

Figure 3.




WARNING: Never charge, short circuit, puncture, deform, incinerate, heat above 85° C, or expose the battery assembly to water. Replace the battery assembly when discharged or when indicated by the AED Plus unit. Refer to the AED Plus Aviation Administrator's Guide for information on the AED Plus unit's self tests and on how to determine the condition of the AED Plus Battery Assembly.

Battery type: LiMnO₂, 2.8 Ah

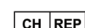

Total Lithium Content: 5.6 g

ZOLL

9652-000857-01 Rev. C

 ZOLL Medical Corporation
269 Mill Road
Chelmsford, MA USA
01824-4105

 ZOLL International Holding B.V.
 Einsteinweg 8A
6662 PW Elst
Netherlands

 ZOLL Medical Switzerland AG
 Baarerstrasse 8
6300 Zug
Switzerland

